

Specimen Requirements

Resolve mdx is an advanced test that combines PCR technology with a unique susceptibility testing method to help physicians select the appropriate treatment for patients with UTI and/or STI symptoms.

Patient Clinical Information Requirements:

- First name and last name
- Date of birth
- Sex (M/F)

Urine Specimen Requirements:

- Clean catch, midstream urine
- Catheter urine
- If ordering both UTI and STI tests, two separate tubes must be submitted

Labeling Instructions:

- First name and last name
- Date of birth

Turnaround Time:

UTI

- Within 24 hours (Negative report – no pathogens detected)*
- Within 48 hours (Positive report – including antibiotic susceptibility testing results)*

STI

- Within 24 hours (Negative or Positive report)*

*From specimen receipt at lab. If shipped as instructed. If both tests ordered, reports will be released concurrently.

Patient Eligibility:

- Patients experiencing symptoms associated with UTI and/or STI, including, but not limited to, dysuria, increased frequency, urgency, foul-smelling urine, cloudy or dark urine, pelvic pain, unusual discharge

Causes for Specimen Rejection:

- >5 days from collection date with C&S preservation tube
- >2 days from collection date in urine cup without preservatives and on ice
- Urine cup on ice: if ice melted, specimen will be rejected
- Urine sample leaked in transit
- Less than 3mL of urine received
- Frozen specimen
- Foley catheter tips
- Collection date not provided



Xpert One Support is founded on personal attention, responsiveness, and commitment to customer satisfaction. Our Xperts are truly experts on mdxhealth products and processes, from specimen requirements and shipping logistics, to clinical insights. Together with your mdxhealth Molecular Diagnostic Specialist, Xpert One Support strives to make your mdxhealth experience the best it can be.

Physicians, patients, lab technicians, and everyone in the fight against prostate cancer—you can expect more from Xpert One Support.



866-259-5644

(Monday – Friday, 5:00 AM - 5:00 PM PT)



info@mdxhealth.com

Joseph L. Sailors, MD
Laboratory Director
CLIA# 45D2229819